



Free Call No.: 1800 451 446 Fax No.: (02) 9638 2700
E-Mail: helpdesk@denyer.com.au Website: www.denyer.com.au

THE SILENT KNIGHT PILLCRUSHER

INSTRUCTIONS FOR USE

- Step 1** Remove a pouch from the dispenser. Place the tablet(s) in the plastic pouch. It is **imperative** that only soft tablets such as panadol are crushed 4-5 at a time. Hard tablets must be crushed individually.
Note: Do not overfill pouches



- Step 2** Lift the handle and place the pouch about half way into the crushing area area. Line up the tablets with the top of the metal crushing plate.
Note: The supplied pouches only must be used or damage to the pillcrusher can result. Furthermore, the usage of other containers may puncture or rip that container resulting in loss of medication.



- Step 3** Without using undue force, press the handle down to crack the pills. **Repeat** pressing the handledown while feeding the tablets further into the crushing area.



- Step 4** Shake the pouch. Put the tablets all the way down to the bottom of the crushing area to complete the crushing.



- Step 5** Tapping the handle rapidly while removing the pouch will pulverize the tablet(s) into a fine powder form.



- Step 6** After reaching the desired particle size, pour the medication into a medicine cup for serving to the patient. Tap the sides and corners of the pouch to remove any trapped particles.



CLEANING & MAINTENANCE

The Silent Knight is made entirely of non-corrosive metal and can be cleaned with any common disinfectant or simply with water. There should be no tablet residue on the pillcrusher if the correct pouches are used and they are not overfilled.